
OHROPAX® Music

EN 352-2:2020, EU 2016/425

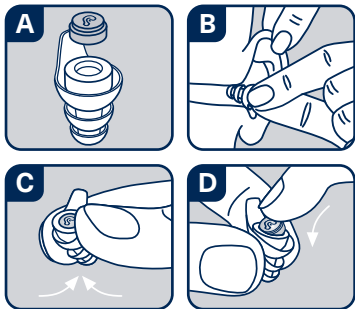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EN Reusable earplugs against harmful noise

OHROPAX® Music – Earplugs with special filter (yellow filter) for an even sound attenuation.

The earplugs consist of two parts: the coloured, round filter and the transparent earplug (image A). The filter already comes delivered inside the larger earplugs (M: Ø 7 – 12 mm). First, insert these earplugs into your ear canal in accordance with the instructions. If this is difficult or feels uncomfortable, try again with the slightly smaller earplugs (S: Ø 6 – 11 mm).

Instructions for use (see picture): Use one hand to pull the outer ear slightly outwards and upwards (image B), and with the help of the tongue, carefully insert the earplug into the ear canal until it is in a secure and comfortable position.

Changing the filter: To remove the filter, press the earplug just under the filter until it „pops“ out of the earplug (image C). To insert the filter, ease it into the larger opening of the earplug with the ear symbol facing outwards. The filter should be inserted at a slight angle and upside down. Then, push the ear symbol downwards a little to ensure the filter is correctly

positioned inside the hollow area (image D). The filter can only provide the certified noise-reducing effect if it is inserted correctly!

Further information: • Insert the earplugs as described and check that they fit correctly. • Ensure hygienic use. Your hands, ears and earplugs should be clean and dry. • The earplugs should be regularly inspected for usability as well as for the necessity of cleaning. • Cleaning: Light impurities can be carefully wiped off with a damp cloth. For a deeper clean with soap or hygiene tablets, remove the filter first. Rinse thoroughly and wait until completely air-dried. • Wear earplugs at all times in noisy environments. • **WARNING:** Sudden or fast removal of the earplug from the ear canal may damage the eardrum. • Failure to comply with these instructions may significantly impair the protective effect of the product. • This product may be adversely affected by certain chemical substances. Further information can be sought from the manufacturer. • Keep away from small children (danger of swallowing). • Can be used by children aged six and older; use only under adult supervision.

Storage: The plugs should be stored in the original box, dry and protected from heat.

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- A** CHEMOMEDICA GmbH, 1010 Wien
OHROPAX GmbH