
# Thermometer flexible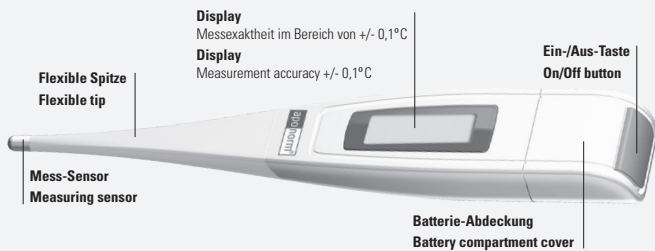

technology by  
microlife

## DE

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## EN

### Important safety instructions

- The thermometer is suitable exclusively for measurement of the bodytemperature!
- The minimum measurement time until the signalling tone [beep] must be maintained without exception!
- The thermometer contains small parts (battery, etc.) that can be swallowed by children. Therefore never leave the thermometer unattended to children.
- Protect the thermometer from hitting and striking!
- Avoid bending the tip more than 45°!
- Avoid ambient temperatures above 60 °C. NEVER boil the thermometer!
- Use only the commercial disinfectants listed in the section «cleaning and disinfecting» to clean the thermometer. The thermometer must be intact when immersed in liquid disinfectant.
- The manufacturer recommends to verify the accuracy by an authorised laboratory every 2 years.
- The function of this device may be compromised when used close to strong electromagnetic fields such as mobile phones or radio installations and we recommend a distance of at least 1 m. In cases where you suspect this to be unavoidable, please verify if the device is working properly before use.

Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

### Turning on the thermometer

To turn on the thermometer, push the button over the display window; a short signalling tone [beep] signals «thermometer ON». A display test is performed. All segments should be displayed. Then at an ambient temperature of less than 32 °C, an «--» and a flashing «°C» appear at the upper right of the display. The thermometer is now ready for use.

### Using the thermometer

Choose the preferred measuring method. During a measurement, the current temperature is displayed continuously. The «°C» symbol flashes during this period of time. If the signalling tone sounds (beep-beep-beep 10 times) and the «°C» is no longer flashing, this means that the measured increase in temperature is now less than 0,1 °C in 6 seconds and that the thermometer is ready for reading. To prolong the life of the battery, turn it off after use by pressing the operating button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

### Storage of measured values

If you hold the button down for more than 3 seconds when turning the thermometer on, the maximum temperature stored automatically at the last measurement will be displayed. At the same time, a small «M» for memory will appear at the right of the display. After 3 more seconds, this value will disappear and the thermometer will switch to the normal measurement mode. Then the stored value is erased.

### Measuring methods

#### In the mouth (oral)

Position the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The measuring sensor must be in good contact with the tissue. Close your mouth and breath evenly through the nose to prevent the measurement from being influenced by inhaled/exhaled air. Approx. measuring time: 30 seconds!

#### In the anus (rectal)

This is the most reliable measuring method, and is especially suitable for infants and small children. Carefully insert the measuring sensor of the thermometer 2 to 3 cm into the anal aperture. Approx. measuring time: 30 seconds!

#### In the armpit (axillary)

To receive more reliable results apornorm® by microlife recommend to measure temperature orally or rectally. A minimum measurement time of 3-5 minutes is recommended although the acoustic signal could ring out even earlier.

### Cleaning and disinfecting

Name: Isopropyl alcohol 70 %.

### Water resistant

You can steep the thermometer into water or Isopropyl alcohol (70 %) for a maximum time of 24 hours.

### Battery replacement

When the «▼» symbol (upside-down triangle) appears in the lower right of the display, the battery is exhausted and needs replacing. For the battery replacement please remove the compartment lid of the thermometer. Replace the exhausted battery by positioning the + at the top. Make sure a battery of the same type is on hand. Batteries can be purchased at any electrical goods shops. For battery disposal please refer to «safety instructions».

| Typ                           | Technical specifications  |
|-------------------------------|---|
| Measurement range             | Maximum thermometer<br>32,0 °C bis 42,9 °C Temp.<br>< 32,0 °C: display «L» for low (too low)<br>Temp. > 42,9 °C: display «H» for high (to high) |
| Measurement accuracy          | ±0,1°C between 34 °C and 42 °C  |
| Operating conditions          | 10 °C to 40 °C  |
| Self-test                     | Automatic internal check at a test value of 37 °C; if there is a deviation of > 0,1 °C, «ERR» (error) is displayed                              |
| Display                       | Liquid crystal display (LCD) with three digits.<br>Smallest unit of display: 0,1 °C   |
| Signalling tone               | For signalling that the thermometer is ready to use and that the temperature increase is less than 0,1 °C / 6 seconds                           |
| Memory                        | For storing the last measured value   |
| Storage conditions            | temperature: -25 °C to +60 °C,<br>humidity: 15 % to 95 % (noncondensing)  |
| Battery                       | 1,5 / 1,55 V; LR 41   |
| Battery lifetime              | approx. 4500 measurements (using a new battery)   |
| IP Class                      | IP67  |
| Reference to standards        | EN 12470-3, clinical thermometers; ASTM E1112; IEC 60601-1; IEC 60601-1-2 (EMC); IEC 60601-1-11   |
| Electromagnetic Compatibility | Device fulfills the stipulations of the standard IEC 60601-1-2  |
| Expected service life         | 5 years or 10000 measurements   |

### Guarantee

We grant you a warranty of 30 years after the date of purchase. Any damage caused by improper handling shall not be covered by the warranty. Batteries and packaging are also excluded from the warranty. All other damage claims excluded. A warranty claim must be submitted with the purchase receipt. Please pack your defective thermometer well and send with sufficient postage to:

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WEPA Apothekenbedarf GmbH & Co. KG, 56204 Hillscheid (Germany),