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INSTRUCTIONS FOR USE Cooling support and relief bandage

Aktimed SPORT is a moist, cooling, semi-rigid support and relief bandage from the group of zinc paste bandages. The formula contains, among other ingredients, a hydrocolloid gel in which arnica montana 10⁻⁶, rhus toxicodendron 10⁻⁸ and ruta graveolens 10⁻⁶ are included. The bandage's main physical/mechanical effect is supported by this formula. The formula is free of fats such as vaseline and of paraffins and is hypoallergenic as well as latex-free.

INSTRUCTIONS FOR USE

- Open wounds/skin areas must first be covered with a sterile wound dressing.
- The bandage should be applied with light tension. With 3 to 4 layers on top of each other, the pleasant cooling effect lasts for several hours.
- A protective bandage on top of the dressing protects against clothes getting soiled.
- The dressing dries within approx. 4 to 5 hours, then forms a stable, compressive shell and can remain in place for up to 6 days.

SAFETY INFORMATION

- The bandage may only be used on open wounds in combination with, and over, a sterile wound dressing.
- Do not use in cases of peripheral arterial occlusive disease, vein inflammation (septic phlebitis), deep vein thrombosis (phlegmasia coerulea dolens), or heart failure, over super-infected wounds, or if you have diabetes.
- Do not use if you are allergic to any of the ingredients (e.g. phenoxyethanol).
- If itching, persistent redness of the skin or blistering occurs, treatment should be interrupted and the doctor in charge should be consulted.
- Keep out of reach of children.

STORAGE INSTRUCTIONS

The bandage should be stored at temperatures of up to max. 35° C. The cooling effect of the bandage can be improved when stored in a cool place. The bandage should be stored horizontally.

COMPOSITION

Fabric: rayon, polyamide. / **Formula:** zinc oxide, water, glycerin, hydrocolloid gel, arnica montana (dilution level 10⁻⁶), rhus toxicodendron (dilution level 10⁻⁸) and ruta graveolens (dilution level 10⁻⁶), phenoxyethanol, dextrin.

PRODUCT RANGE

AW 20123 - single-pack bandage (8 cm x 5 m)
incl. protective bandage (7,5 cm x 4,5 m)

CE: conforms to EU Directive 93/42/EEC for medical products
Production: Germany / **Last updated:** 09/2018
Aktimed GmbH, Theaterstraße 18, 69117 Heidelberg, Germany
www.aktimed-tape.de

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Aktimed GmbH, Theaterstrasse 18, 69117 Heidelberg, Alemania
Aktimed GmbH, Theaterstrasse 18, 69117 Heidelberg, Germania