
Cellacare® Dotop 21 Comfort

Rückenorthese
Ceinture de soutien lombaire
Back orthosis
Ôrtesis de soporte lumbar
Ortesi dorsale
Rugorthese

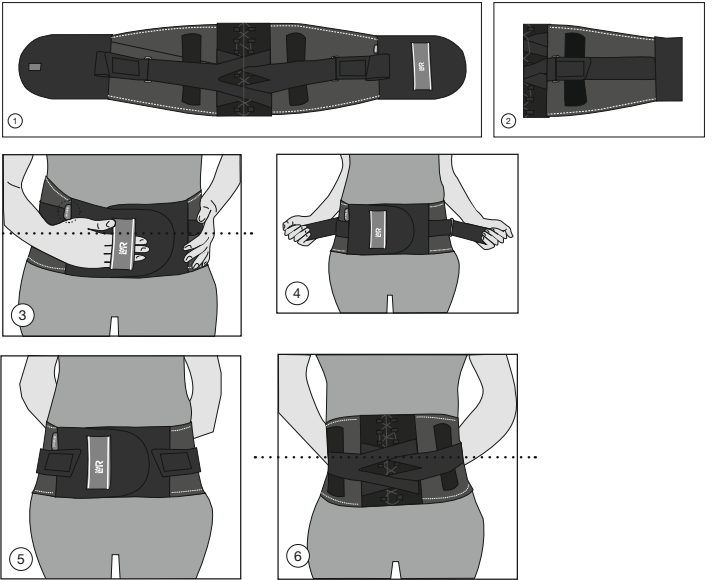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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions,
please contact your physician or specialist shop.

Product description and performance characteristics

Cellacare Dotop 21 Comfort is a back orthosis with stabilizing and relieving effect. 6 stays, 4 steel stays in the back adaptable and 2 supple stays in the front, and a 3 points strap system help to support and relieve the lumbar spine.

Product composition:

Polyester, elastane, polyamide, polyurethane, polyethylene, acrylic resin, acier, polypropylene, alloy of polyolefins, cotton, viscose, Zamak, elastomer.

Intended purpose

Orthosis for relief and stabilisation of the lumbar spine.

Indications

- Acute or chronic nonspecific lumbar spine pain syndrome, with muscle or ligament pain of the lumbar spine and sacrum area
- Acute or chronic specific lumbar spine pain syndrome in the case of:
 - Osteochondrosis in the lumbar spine area
 - Intervertebral disk protrusions or prolapses in the lumbar spine
 - Spondylarthritis in the lumbar spine area
 - Spondylolysis with or without spondylolisthesis (grade I)
- Postoperative lumbar support after intervertebral disk surgery

Contra-indications

- Known allergy and / or hypersensitivity to any of the product components
- Severe back injuries / disorders such as instability / fragility of the thoracic, lumbar spine (with vertebral fractures) and/or pelvis in case of bone tumours or metastases or advanced osteoporosis

Side effects

In very rare cases, skin irritations and / or allergies may occur.

Warnings and precautions

- Do not use Cellacare Dotop 21 Comfort unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare Dotop 21 Comfort shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before use.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of pregnancy, the patient's treating physician needs to give the approval before using the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- Cellacare Dotop 21 Comfort must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare Dotop 21 Comfort is intended for use on a single patient only.
- The product's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example when the product is not being worn or when being washed.

- Cellacare Dotop 21 Comfort must not be used beyond the last day of the month indicated on the packaging (expiry date).
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information

Preparation

- Apply Cellacare Dotop 21 Comfort while standing.
- For hygienic reasons wearing a shirt underneath the orthosis is recommended.
- Open the front hook-and-loop fastener and flatten out the orthosis in front of you (Fig. 1).
- Before applying, make sure that the ends of the straps are fastened as close as possible to the metal buckles in order to have the greatest length of strap (Fig. 2).

Usage

Hand grips in the right hand, Cellacare label at the top

- Apply the orthosis around the waist. Fasten the belt, left side covered by right side. (Fig. 3).
- Check the fit of orthosis. If required, increase or reduce the pressure by pulling harder or less hard when closing the orthosis.
- Grip both ends of the tightening straps at the same time. Open the hook-and-loop fasteners and pull the straps to the front until a pleasant and stabilising pressure occurs (Fig. 4). Attach the ends of the straps to the front of the orthosis by using the hook-and-loop fasteners (Fig. 5).
- Check the fit of the orthosis (Fig. 6). If required, increase or reduce the pressure by pulling harder or less hard when closing the tightening straps.

Care recommendations

Machine wash the orthosis at 30 °C (delicate cycle) with like colors. Do not bleach. Allow Cellacare Dotop 21 Comfort to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Fasten the hook-and-loop closures and if possible use a mesh bag. Clean the hook-and-loop closures regularly for they retain their ability to fasten tightly, and to preserve the performance of the product.

Storage and transport conditions

Cellacare Dotop 21 Comfort must be kept away from light and moisture (preferably in the original packaging).

Disposal

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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