Evolsin® Hydrokolloid Rolle

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Evolsin® Hydrocolloid Roll

The EVOLSIN® HYDROCOLLOID ROLL is a gelatine-free hydrocolloid roll. The elastic hydrocolloid film serves to cover and protect stressed areas of skin and to absorb exudates.

Purpose (intended use)
The EVOLSIN® HYDROCOLLOID ROLL is intended to be used as a mechanical protective cover for superficial areas of skin.

The product is used to:

- cover and protect the affected skin area;
- shield against external mechanical influences (e.g., friction, pressure); and
- absorb exudate.

Areas of application include:

- Blisters
- Corns
- Calluses
- Bunions
- Superficial skin blemishes
- Superficial skin irritation in the lip, nose or chin area

Contraindications
DO NOT use on injured skin!
The product must not be used in the event of a known hypersensitivity to any of the ingredients, particularly polyisobutylene or carboxymethyl cellulose.
Precautions
1. Do not use on open or infected skin.
2. Do not use on mucous membranes.
3. People with diabetes or circulatory disorders should seek medical advice before use.
4. In the event of persistent skin changes, increasing redness or swelling, discontinue use and seek professional advice.

5. The plaster is for single use only.
6. Do not use at the same time as creams, ointments or lotions, as this may affect adhesion.
7. Apply the plaster before applying cosmetic products.
8. Do not use the product if the packaging is damaged or open.
9. Do not use on wounds or burns.

Instructions for use

1. Clean the affected area of skin with soap and water and dry it thoroughly.
2. The plaster comes on a roll and can be cut to size as required.
3. Remove the protective film and place the plaster centrally on the affected site.
4. Carefully smooth the plaster down onto the surrounding skin.
5. The plaster should remain on the skin until it begins to come off on its own.
6. If necessary, apply a new plaster.

Removing the plaster

1. Carefully lift the edges while, at the same time, stretching the skin slightly downwards.
2. Slowly peel the plaster off the surface of the skin.
3. Alternatively, slight horizontal stretching can facilitate removal.
4. Dispose of the used plaster properly.

Replacement interval

1. The plaster should remain on the skin for as long as possible.
2. If the adhesive wears off, the plaster needs to be changed.

 EVOLSIN® HYDROCOLLOID ROLL is a medical device.

 Evolsin medical UG (haftungsbeschränkt), Schlehenweg 7, 61267 Neu-Anspach, Germany
www.evolsin-medical.com
Onlineshop: **www.evolsin-medical.com** - Questions?: **info@evolsin-medical.com**

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